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MidLiner Use Instructions:

- Position baby in supine midline
• Open Velcro® front and loosen adjustable colored side tabs
• Slide back of NICU Midliner under baby's head with the support rolls resting flat on the bed surface
• Close Velcro front opening and adjust Velcro side tabs for a snug fit.

For Supine Midline:

When baby is ready to follow a positioning plan:

For Side Lying and Prone:

- Simply tuck the lower support roll under infants head and turn the whole baby into side lying or prone position.
• Make sure both rolls are positioned posterior to baby's head.

- Respiratory devices and other cannulas should remain in place.

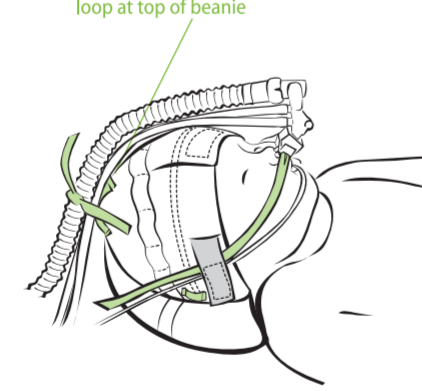
For Skull Molding:

- Open Velcro front and loosen side tabs.
• Turn baby's head to desired position while keeping Midliner and side rolls flat on bed.
• Close Velcro front opening and adjust side tabs to snug fit.
• You should now have one roll positioned in front of the baby's head and one roll positioned posterior to baby's head.
• Adjust respiratory equipment if needed.

For Nasal CPAP:


- Attach nasal CPAP side ties under the Velcro in the colored side tabs or by using the lower loops as shown
• For securing the CPAP tubing, use the ties included in the package to attach to the white loops on the midliner

Fisher-Paykel – Use one midline loop at top of beanie

For Nasal Cannulas or Feeding Tubes:

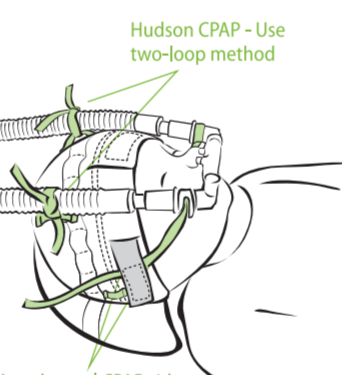- Attach with or without tape using the colored Velcro side tabs. Make sure you have a snug fit.
• Nasal cannulas should be cinched over the top of the beanie as shown in the diagram

Attach nasal CPAP side ties under Velcro tab or to lower loop

Use side tabs to hold nasal cannulas and feeding tube

Laundry Instructions:

The Midliner can be hand washed in cool water, and air dried, or can be washed in residential washing machines and dryers.

QTY: 1

- Cotton blend with Lycra - High resilience foam
• Non-Prescription - Latex free - Phthalates free - Formaldehyde free
• Single Patient Use
• X-ray friendly

For detailed positioning plan, see www.tortlemedical.com

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Tortle Products LLC
Tortle Products LLC, 1444 Wazee Street