Regular: OFC 33-38 cm (13-15 in.)

Instructions for use:

1. Disconnect the Velcro.
2. Position the eye pads over the eyes.
3. Pass the lower strap under the small of the neck and attach velcro.
4. Secure the Velcro of both straps and position the EyeMax2™ as shown.
5. Make sure the strap completely covers the ears.
6. If the eye patch rides up away from the nose, tighten the lower strap.

Catalog # R300P01 **Regular**: occipital-frontal circumference **33-38** cm in packs of 20.
 Catalog # R300P02 **Preemie**: occipital-frontal circumference **26-32** cm in packs of 20.
 Catalog # R300P03 **Micro**: occipital-frontal circumference **20-25** cm in packs of 20.

2 **Single Patient Use Only**
Non-Sterile, Latex-Free

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Precautions:

- Caution** ⚠ Do not tug on the EyeMax™ to reposition it. A premature infant's skin is very delicate and can easily rupture.
- Caution** ⚠ Avoid obstructing the nose. Check regularly to ensure the EyeMax™ is not blocking the patient's airway.
- Caution** ⚠ EyeMax™ is not a bandage. Cover any existing skin abrasions with appropriate bandages before use.
- Caution** ⚠ Do not over tighten – causing excessive pressure on the eyes.
- Caution** ⚠ Be sure the strap is covering the ear, not hooked over the ear, and that the eye pad is not folded over.
- Caution** ⚠ Replace the EyeMax™ with a new one if it becomes over-stretched.
- Caution** ⚠ Reuse of this device could cause cross-contamination or product malfunctions.

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Maxtec Inc., 6526 South Cottonwood Street, Salt Lake City, Utah 84107