

Bili✓Chek

Quick Reference



PEOPLE PRODUCTS PROGRAMS



Press top gray Function Button to turn on unit.



Snap tip onto unit.



Press Trigger Button once to began calibration, again to end.



Gently press tip to forehead until light is green, wait for beep. Repeat steps 6 & 7 four more times.

1. Press the top gray Function Button to turn the unit on (figure A). The unit will beep once when the self test is completed.
2. Snap tip onto Bili✓Chek (figure B). Press and release the Trigger Button to begin calibration (figure C). Observe the three dashes (---) indicating ready to calibrate.
3. Press and release Trigger Button (figure C) again to complete calibration (005 will be displayed). The unit is ready to take 5 measurements.
4. Remove tab from the disposable tip.
5. Press and release Trigger Button (figure C).
6. Gently press the Bili✓Chek tip onto the baby's forehead until the light turns green. Hold the unit in place with steady pressure until unit beeps and displays (004) (figure D).
7. Remove unit from infants forehead and repeat steps 5 and 6 four more times until a final measurement is obtained. (Be sure to release the pressure applied to infants forehead between each measurement.)

★ If an error message occurs during measurement refer to reverse side of this card. Clear the error and resume measurement.



INFANT CARE PROGRAM

BiliChek

Error Code Quick Reference



PEOPLE PRODUCTS PROGRAMS

- E00** Setup information is not entered, refer to users manual.
- E01** Calibration failure, remove tip and replace, try again. Replace tip if necessary.
- E02** Erroneous measurement, clear error and try again. Replace tip if necessary.
- E03** Tip removed in the middle of measurement. Start over with new tip.
- E04** Too much pressure on baby's forehead, clear error and try again.
- E05** Reading too High, do a heel stick.
- E10** Light bulb burned out, replace bulb.
- E99** System error, call for service.

Respironics • 1001 Murry Ridge Lane • Murrysville, Pennsylvania USA • 800-345-6443 USA • +01-724-387-4000
Respironics Deutschland • Gewerbestrasse 17 • 82211 Herrsching Germany • +49 8152 93060



101549
HM 4/30/0