

## **MEDI-SOL ADHESIVE REMOVER WIPES AND LIQUID**

### **SAFETY**

Medi-Sol Adhesive Remover Wipes and Liquid is a powerful and safe citrus-based adhesive remover specially formulated for medical applications. Medi-Sol is non-carcinogenic and contains no harsh solvents. It is also non-irritating, non-toxic and biodegradable.

### **USAGE**

To use Medi-Sol, apply directly to edge of tape or appliance while gently lifting until completely removed from the skin. Wipe off any remaining residue and follow with a gentle soap and water solution for final cleaning. To remove tar and asphalt from the skin and mucous membranes, use a liberal application and gentle wiping to remove. Eye injuries can be cleaned by copiously irrigating the eye with Medi-Sol saline solution. To use Medi-Sol Adhesive Wipes, swab the area to be cleansed or wipe gently over the edge of tape or appliance while lifting gently until completely removed.

### **APPLICATIONS**

Medi-Sol is used for adhesive tape, ostomy appliances, TENS accessories, nitro-patches, adhesive residue, tar removal and hot-melt removal. This product can be used safely on medical appliances and equipment to remove tapes and adhesives as well as on the patients' skin.

### **AVAILABLE**

Available in 4 ounce, 16 ounce and 1 gallon plastic bottles. The Wipes are available in single use wipes; 100 per box, 600 per carton and 1,000 per case.